PATIENT CARE INSTRUCTIONS

FP3D™ material is engineered to adapt to intraoral conditions. Avoid aggressive flexing outside of its intended use.



Clean the appliance with soap and warm water, or any over-the-counter mild cleaning agents indicated for oral devices.



If not wearing, dry partial dentures thoroughly overnight; do not leave submerged in the cleaner.



DO NOT soak partial dentures overnight, as this may cause discoloration.

Carbon



MKT-OH-FP3DPATIENTCAREINST-GEN-ONEPGR_0925